

Instructions for Rapid at Home Antigen Covid-19 Test


PLEASE READ THESE INSTRUCTIONS COMPLETELY BEFORE STARTING THE COLLECTION PROCESS.

This kit is a rapid antigen test. Results will be available in 15 minutes. This is NOT a PCR test and does not take the place of a PCR test. It is recommended by health care professionals that regardless of the results of this test, that if you are symptomatic and/or feel you need a test, to follow this test with a PCR test. Contact your health professional for testing locations and availability of the PCR test.


If your COVID-19 test is positive, please take a picture of the kit showing the results and save it in case this information is required by NWACC. Please refer to www.nwacc.edu/coronavirus for updated information on how to report positive COVID-19 results.

STEPS:

- 1) Please lay out the 4 items in the test kit.
- 2) Take the nasal swab and insert into one nostril (approximately 1 inch) and slowly swirl the swab 5 times. Remove the swab and insert into the other nostril (approximately 1 inch) and slowly swirl the swab 5 times. Slowly remove the swab from the second nostril while swirling. This step should take approximately 15 seconds to complete.

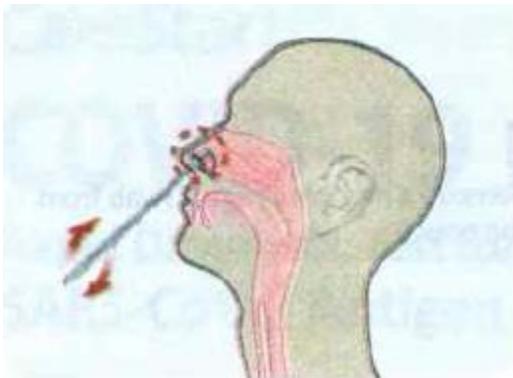


- 3) Remove the foil seal from the extraction vial.

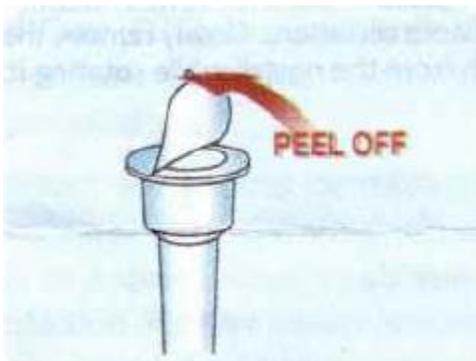


- 4) Place the swab into the extraction vial and make sure that the swab is in the liquid at the bottom. Vigorously swirl the at least 5 times.



- 5) Remove the swab by swirling the swab against the sides of the extraction vial while squeezing the vial to release the liquid from the swab.

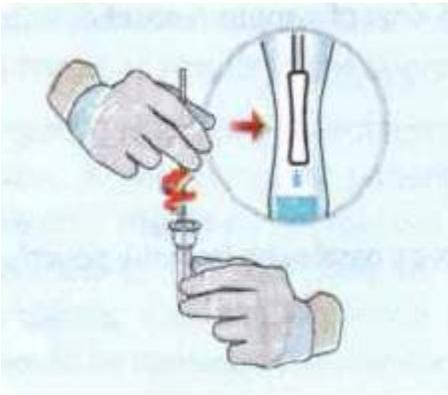


- 6) Place the cap onto the vial. Push down firmly on the cap to make sure it is securely in place.

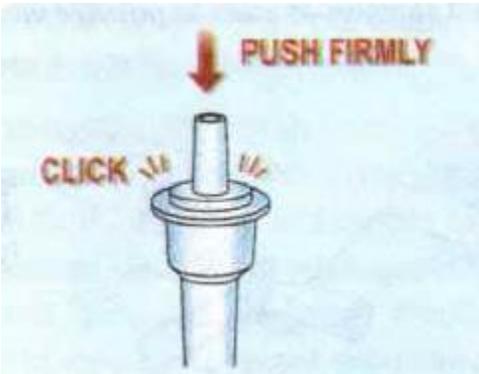


7) Mix thoroughly by flicking the vial.



8) While holding the vial upside down at 90 degrees, squeeze the extraction vial and squeeze 3 drops of liquid into the sample well.

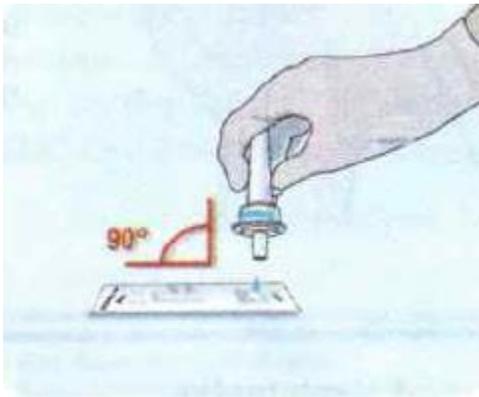


9) Read and interpret results at 10 minutes and no more than 15 minutes. After 15 minutes, the results are not valid.

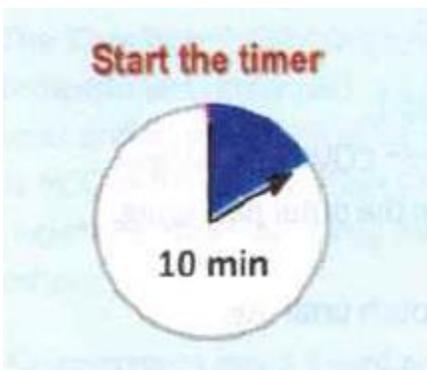


Interpreting your results:


Positive results: Two distinct lines. One red next to “C” and one blue next to “T”.
Even if the line is light, a visible line on the T line is a positive results

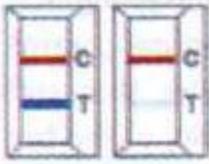

Negative results: One red colored line next to “C”.

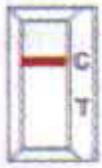

Invalid results: No red line next to the “C”. Even if the “T” line is visible (or no lines), if there is no red line, the test is invalid.

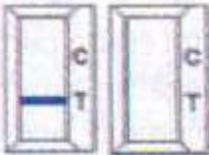

If your COVID-19 test is positive, please take a picture of the kit showing the results and save it in case this information is required by NWACC. Please refer to www.nwacc.edu/coronavirus for updated information on how to report positive COVID-19 results.